


TytoHome Quick Start Guide

To access **Connected Care**

How to pair your TytoHome device with Connected Care to start your virtual consultations


Log in or register

As a Discovery client, you will not need to register. Use your Discovery login details.

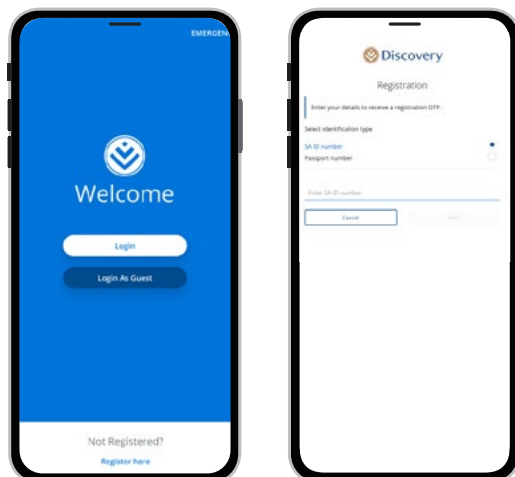

Pair your device


Select 'Add Device' to connect your TytoHome device

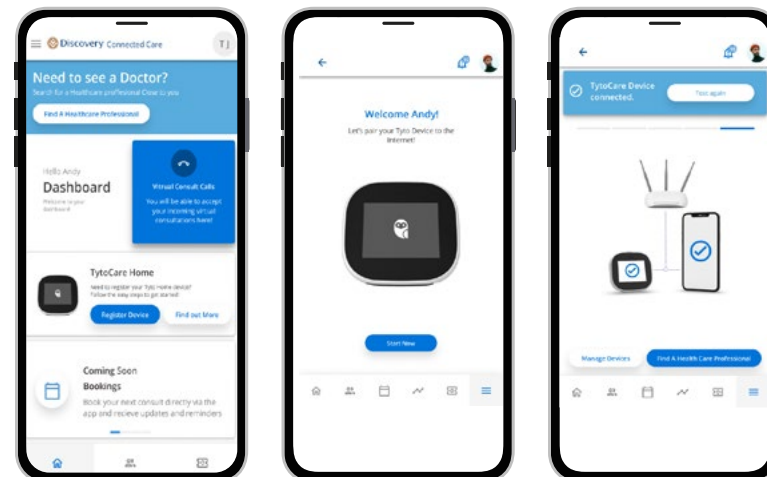


Make sure you have an internet connection, select 'add device', and follow the on-screen instructions.

Find a doctor and book a consultation

You can find a doctor and book a consultation.

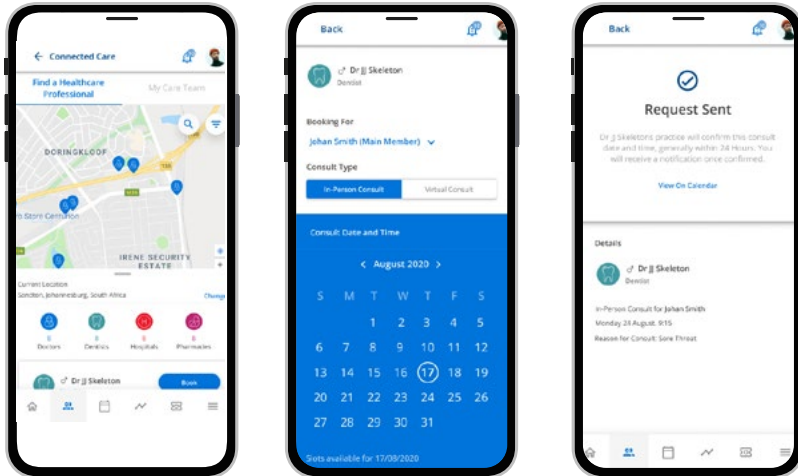

Use your TytoHome device

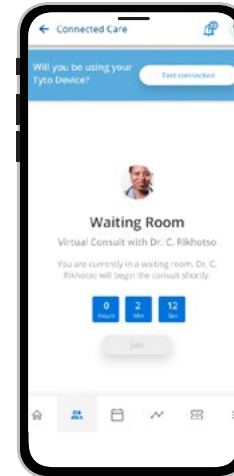

Use your TytoHome device during your online consultation so your doctor can conduct a fully guided medical examination.